

120*200mm说明书 正背面印刷105g铜版纸

Disposable Dental Air Water Syringe Tips
Instruction For Use



Huanghua Promisee Dental Co., Ltd.
Mould Industry Garden, Empolder Section,
Huanghua, 061100 Hebei,P.R.China
Tel:+86-317-8882702 FAX:+86-317-8881708

EU REP MedNet EC-REP GmbH
Borkstrasse 10, 48163 Muenster , Germany

Product Name: Disposable Dental Air Water Syringe Tips
Brand: COTISEN
AWT01, AWT02, AWT03, AWT04, AWT05

Product Description

As the necessary consumer goods of dental chair, disposable dental air water syringe tips are essential fittings of dental treatment. It's generally composed of internal & external casing or independent plastic pipe.

AWT01 is composed of white plastic tube and stainless steel inner pipe, internal &external casing pipe structure, type of steam-water separation.
AWT02 is composed of independent white plastic tube, independent channel, type of none steam-water separation.
AWT03 is composed of clear tube and colored inner pipe, internal &external casing pipe structure, type of steam-water separation.
AWT04 is composed of clear tube and colored inner pipe, with same structure as AWT03, just shorter than AWT03.
AWT05 is made of a transparent independent plastic pipe, with seven same inner holes in a multi-tube structure.

Intended Use

The disposable dental air water syringe tips are used to be attached to the air water syringe unit so as to deliver a controlled spray of air and water during dental procedures.
It's used for cleaning and drying mouth and teeth by syringing water or/and blowing air during dental treatment. It can only syringe water and/or air to the mouth. It is provided in non-sterile, the user needs to disinfect the device before use and it can be used only once.

Intended User

The intended users are Professional medical staffs, includes physicians, nurses, therapists and other relevant staffs with medical science.

Patient Population

- 1.Age group: non-limited, the person who need dental treatment;
- 2.Weight range: No limit;
- 3.Health: person who have dental disease;
- 4.Country: all.

Indications

The Disposable Dental Air Water Syringe Tips are used in dental unit, by spraying gas and water, the tooth surface and mouth can be cleaned, removing blood, saliva and other impurities, improving the effectiveness of the dental professional and the comfort of the patient.

Limitation

The device can only be used by professional medical staffs. Please use the device within the expire date.

Contraindications

The product is made of PVC materials that containing DEHP. Anybody allergic to DEHP please do not use the device.

Adverse Reactions

Not found.

Usage Method

- a.Disinfect the product before use.
For details, please refer to Processing Instruction before use(Single-Used Medical Devices)
- b.Install the air water syringe tip. Take out the syringe tip, and insert the end with card slot into air water syringe as far as possible.
For details, refer to Installation Instructions as below;
- c.After installation, pull the tip lightly to ensure the syringe tip is securely installed;
- d.Test for proper air and/or water separation, you need to press water &gas button separately or simultaneously.

Model	Installation Instructions			
AWT01 AWT02 AWT03 AWT04				
	a.Take out the air/water syringe tip AWT01, AWT02, AWT03, AWT04 and air/water syringe as shown.	b.Press the metal clip ring of outer ring.	c.Insert the tip into the connecting interface as picture shown. When the metal clip is pressed down until all the groove enter the connecting interface and can't move forward any more, then, release the metal clip.	d.Press the metal clip again when you need to replace a new tip.

Model	Installation Instructions			
AWT05				
	a.Take out the air/water syringe tip AWT05 and air/water syringe as shown.	b.Unscrew the tapered metal clip in the direction indicated in the picture above.	c.As shown in the picture above, insert the tip into the air/water syringe through the conical metal ring with normal force until it can no longer be inserted.	d.Then tighten the tapered metal ring (the direction of tightening is shown in the figure above).

Warning and Precaution

- 1.The product is only used by qualified medical staff.
- 2.It's forbidden to use when the products are deformed or damaged. And it cannot be used when the package is damaged.
- 3.The disposable tips are not autoclavable. Please dispose after use according to local regulation required.
- 4.It is provided in non-sterile, the user needs to disinfect the device before use.
- 5.Use a pair of tweezers or a gloved hand to take the tip to avoid cross-infection. The tip cannot touch the hair or the human body. If the tip falls to the ground, it must be discarded.

Storage Condition

- 1.This product should be stored in a clean, dry, ventilated, room temperature, relative humidity not greater than 80%, and no corrosive gas.
- 2.Avoid direct sunlight.
- 3.Prevent external extrusion and deformation during transportation and storage.

Shelf Life

3 years.

Symbols

CE 0197	CE certified by TÜV Rheinland		Consult instructions for use	#	Model number
EU REP	European union representative		Non-sterile	↑↑	Keep upwards
DEHP	Contains or presence of phthalate	MD	Medical device		Use-by date
	Caution	LOT	Batch code		Made In China
	Keep dry		Date of manufacture		Do not re-use
	Keep away from sunlight		Manufacturer		Importer
ISO 13485	ISO13485 certification	UDI	Unique device identifier		

Processing Instructions before use (Single -Used Medical Devices)

Manufacturer: Huanghua Promisee Dental Co., Ltd. Device(s): Disposable Dental Air Water Syringe Tips Model: AWT01, AWT02, AWT03, AWT04, AWT05	
INSTRUCTIONS	
Cleaning	Not Applicable. Because it's processing before use for single used medical device.
Disinfection	Soak in 2% glutaraldehyde for 1 hour before use, and then rinse repeatedly with purified water.
Drying	Dried with low-linting wipes.
Storage	1. This product should be stored in a clean, dry, ventilated, room temperature, relative humidity not greater than 80%, and no corrosive gas. 2. Avoid direct sunlight. 3.Prevent external extrusion and deformation during transportation and storage.
Manufacturer contact	Mould Industry Garden, Empolder Section, Huanghua, 061100 Hebei, P.R. China Tel: 0086-317-8882702 www.promiseedental.com Email: info@promiseedental.com

KT/CE01-09-01, Ver. B/2 Released on Feb. 2, 2026