

Disposable Saliva Ejectors
Instruction for Use



Huanghua Promisee Dental Co., Ltd.
Mould Industry Garden, Empolder Section,
Huanghua, 061100 Hebei, P.R. China
Tel: +86-317-8882702
www.promiseedental.com

EU REP MedNet EC-REP GmbH
Borkstrasse 10, 48163 Muenster, Germany

Product Name: Disposable Saliva Ejectors
ST06A ST06B

Brand: COTISEN

Product Description

As dental chair supporting supplies , disposable saliva ejectors is now essential for oral treatment . This product can be independent plastic pipe structure . ST06A / ST06B is a short straight tube in two sizes. The product can be used as the adaptor of all types of saliva ejector. It can be connected with the aspirator tips single or in pairs, then connect with aspirator of dental chair. Which solves the problem that the absorption device can not be used at the same time when the absorption device is busy.

Intended Use

The disposable saliva ejectors are used to be linked to strong/weak suction handle of the dental chair device to deliver controlled fluids during dental treatment. Under the control of suction device, it can effectively absorb blood, saliva and other foreign matter in the patient's mouth. It is non-sterile, users need to disinfect the device before use, and it is single-use.

Intended user

The intended users are professional medical staffs, includes physicians, nurses, therapists and other relevant staffs with medical science.

Patient Population

- 1.Age group: non-limited, the person who need dental treatment.
- 2.Weight range: No limit.
- 3.Health: person who have dental disease.
- 4.Country: all.

Indications

The Disposable Saliva Ejectors is used in dental unit. The disposable saliva ejectors is used in dental unit. Keep the patient's mouth dry by aspirating fluids. Blood, saliva and other impurities can be removed. Improve dental professional productivity and patient comfort.

Limitation

The device can only be used by professional medical staffs. Please use the device within the expire date.
Contraindications: No contraindication is identified by now.
Adverse Reactions: Not found.

Usage Method

(a) Disinfect the product before use.

For details, please refer to Processing Instruction before use(Single-Used Medical Devices)



(b) Installation of disposable saliva ejectors: Take out the saliva ejectors and make sure the strong/weak suction handle matches this specification. Fully insert one end of the long tube into the strong/weak suction handle interface.

For details, refer to Installation Instructions as below

(c) After the installation is completed, gently pull the saliva suction tube to ensure that the disposable saliva ejectors is firmly installed.

(d) Control the switch of the dental chair suction device and the strong/weak suction handle switch valve, and test the suction power of the disposable saliva ejectors and whether the suction is smooth.

Installation Instructions

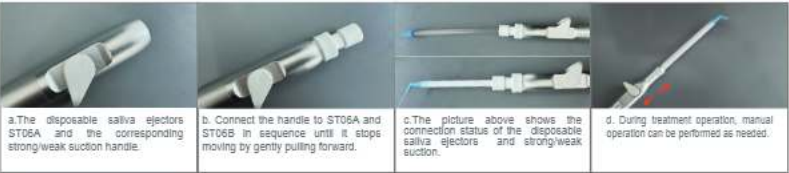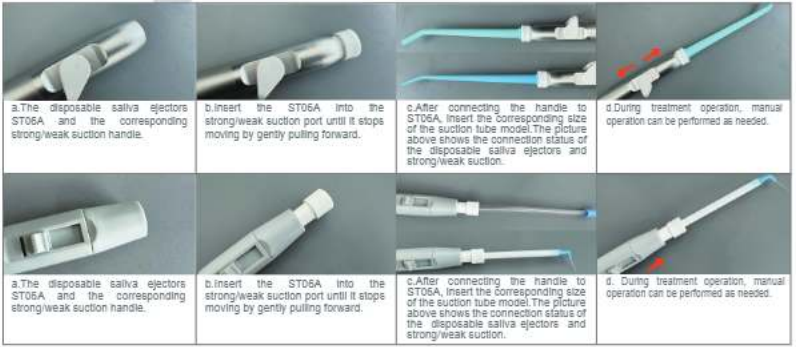

Warning and Precaution

- 1.The product is only used by qualified medical staff.
- 2.It's forbidden to use when the products are deformed or damaged. And it cannot be used when the package is damaged.
- 3.The disposable saliva ejectors are single use. Please dispose after use according to local regulation required.
- 4.It is provided in non-sterile, the user needs to disinfect the device before use.
- 5.Use a pair of tweezers or a gloved hand to take the disposable saliva ejectors to avoid cross-infection. The disposable saliva ejectors cannot touch the hair or the human body. If the disposable saliva ejectors falls to the ground, it must be discarded.

Storage Condition

- 1.This product should be stored in a clean, dry, ventilated, room temperature 10°C - 35°C, relative humidity not greater than 80%, atmospheric pressure 50 kPa - 106 kPa, and no corrosive gas.
- 2.Avoid direct sunlight.
- 3.Prevent external extrusion and deformation during transportation and storage.

Shelf Life

3 years.

Symbols

CE 0197	CE certified by TUV Rheinland	Consult instructions for use	#	Model number
EU REP	European union representative	Non-sterile	11	Keep upwards
Caution	MD	Medical device	Use-by date	
Keep dry	LOT	Batch code	Made in China	
Keep away from sunlight	Date of manufacture	UDI	Unique device identifier	
Manufacturer	Importer	ISO 13485	ISO 13485 certification	

Processing Instructions before use (Single -Used Medical Devices)

Manufacturer: Huanghua Promisee Dental Co., Ltd. Device(s): Disposable Saliva Ejectors Model: ST06A ST06B	
INSTRUCTIONS	
Cleaning	Not Applicable. Because it's processing before use for single used medical device.
Disinfection	Soak in 2% glutaraldehyde for 1 hour before use, and then rinse repeatedly with purified water
Drying	No need.
Storage	1. This product should be stored in a clean, dry, ventilated, room temperature 10°C - 35°C, relative humidity not greater than 80%, atmospheric pressure 50 kPa - 106 kPa, and no corrosive gas. 2. Avoid direct sunlight. 3. Prevent external extrusion and deformation during transportation and storage.
Manufacturer contact	Mould Industry Garden, Empolder Section, Huanghua, 061100 Hebei, P.R. China Tel: 0086-317-8881708 www.promiseedental.com Email: info@promiseedental.com

KT/CE02-09-04, Ver. B/1 Released on Jul. 15, 2025