

Disposable Saliva Ejectors
Instruction For Use



Huanghua Promisee Dental Co., Ltd.
Mould Industry Garden,Empolder Section,
Huanghua,061100 Hebei,P.R.China
Tel:+86-317-8882702
www.promiseedental.com

EU REP MedNet EC-REP GmbH
Borkstrasse 10,48163 Muenster,Germany

Product Name: Disposable Saliva Ejectors
ST07

Brand: COTISEN

Product Description

As dental chair supporting supplies , disposable saliva ejectors' is now essential for oral treatment . This product can be independent plastic pipe structure . ST07 consists of a long tube , a connecting head , and a suction head . The suction head is ultra-fine and is used for rinsing the root canal and absorbing foreign objects during dental root canal treatment .

Intended Use

The disposable saliva ejectors are used to be linked to weak suction handle of the dental chair device to deliver controlled fluids during dental treatment . Under the control of suction device , it can effectively absorb blood , saliva and other foreign matter in the patient's mouth . It is non-sterile , users need to disinfect the device before use , and it is single-use .

Intended user

The intended users are professional medical staffs , includes physicians , nurses , therapists and other relevant staffs with medical science .

Patient Population

- 1 Age group : non-limited , the person who need dental treatment .
- 2 Weight range : No limit .
- 3 Health : person who have dental disease .
- 4 Country : all .

Indications

The Disposable Saliva Ejectors is used in dental unit . The disposable saliva ejectors is used in dental unit . Keep the patient's mouth dry by aspirating fluids . Blood , saliva and other impurities can be removed . Improve dental professional productivity and patient comfort .

Limitation

The device can only be used by professional medical staffs . Please use the device within the expire date . Contraindications: No contraindication is identified by now . Adverse Reactions: Not found .

Usage Method

- a.Disinfect the product before use .
- For details , please refer to Processing Instruction before use(Single-Used Medical Devices)
- b.Installation of disposable saliva ejectors: Take out the saliva ejectors and make sure the weak suction handle matches this specification . (It is recommended to use the standard SE vacuum valve size: 7/32).Fully insert one end of the long tube into the weak suction handle interface .
- For details , refer to Installation Instructions as below;
- c.After the installation is completed , gently pull the saliva suction tube to ensure that the disposable saliva ejectors is firmly installed .
- d.Control the switch of the dental chair suction device and the weak suction handle switch valve , and test the suction power of the disposable saliva ejectors and whether the suction is smooth .

Installation Instructions



a.The disposable saliva ejectors ST07 and the corresponding weak suction handle .

b.Insert the long end of the disposable saliva ejectors into the weak suction port ,until it stops moving by gently pulling forward .

c. The picture above shows the connection status of the disposable saliva ejectors and weak suction .

d.During treatment operation , manual operation can be performed as needed .

Warning and Precaution

- 1.The product is only used by qualified medical staff .
- 2.It's forbidden to use when the products are deformed or damaged . And it cannot be used when the package is damaged .
- 3.The disposable saliva ejectors are single use . Please dispose after use according to local regulation required .
- 4.It is provided in non-sterile , the user needs to disinfect the device before use .
5. Use a pair of tweezers or a gloved hand to take the disposable saliva ejectors to avoid cross-infection . The disposable saliva ejectors cannot touch the hair or the human body . If the disposable saliva ejectors falls to the ground , it must be discarded .

Storage Condition

- 1.This product should be stored in a clean , dry , ventilated , room temperature10°C - 35°C , relative humidity not greater than 80% , atmospheric pressure 50 kPa - 106 kPa , and no corrosive gas .
- 2.Avoid direct sunlight .
- 3.Prevent external extrusion and deformation during transportation and storage .

Shelf Life

3 years .

Symbols

CE 0197	CE certified by TÜV Rheinland		Consult instructions for use	#	Model number
EU REP	European union representative		Non-sterile		Keep upwards
	Caution	MD	Medical device		Use-by date
	Keep dry	LOT	Batch code		Made In China
	Keep away from sunlight		Date of manufacture		Do not re-use
	Manufacturer		Importer	UDI	Unique device identifier
ISO 13485	ISO13485 certification				

Processing Instructions before use (Single -Used Medical Devices)

Manufacturer: Huanghua Promisee Dental Co., Ltd. Device(s): Disposable Saliva Ejectors Model: ST07	
INSTRUCTIONS	
Cleaning	Not Applicable. Because it's processing before use for single used medical device.
Disinfection	Soak in 2% glutaraldehyde for 1 hour before use , and then rinse repeatedly with purified water .
Drying	No need .
Storage	1.This product should be stored in a clean , dry , ventilated , room temperature - 10°C - 35°C , relative humidity not greater than 80% , atmospheric pressure 50 kPa - 106 kPa , and no corrosive gas . 2. Avoid direct sunlight . 3 Prevent external extrusion and deformation during transportation and storage .
Manufacturer contact	Mould Industry Garden ,Empolder Section , Huanghua , 061100 Hebei , P.R. China Tel: 0086-317-8881708 www.promiseedental.com Email: info@promiseedental.com